Intellect[®] **TENS**

DUAL CHANNEL STIMULATOR KIT
A MAJOR STEP IN PAIN MANAGEMENT

INSTRUCTION MANUAL

ISO 13485 Certified

Application of Council Directive(s): **MDD 93/42/EEC**

Class: **Class IIa**

UMDNS: 17191, Electrodes, Transcutaneous Electric Nerve Stimulator

Application of Standard(s): EN 60601-1: 1990 w/ A1, A2, A12, A13; IEC 60601-2-10: 1987; SEMKO PA 826: 1991

Manufacturer's Name:

Chattanooga Group, Inc.
4717 Adams Road
Hixson, Tennessee 37343
USA

Manufacturer's Address:

Representative's Name:

Medical Device Safety Service

Representative's Address:

Burckhardtstr. 1
D-30163 Hannover
Germany

Type of Equipment: Electrode, Transcutaneous Electric Nerve Stimulation

Model:

Intelect TENS

We declare with sole responsibility, that our product **TENS**, meet the essential requirements of Council Directive 93/42/EEC.

Date: 01/26/99

Charles M. Thomas
Managing Director
Chattanooga Group, International Division

02/11/99

Date:

Burckhardtstr. 1

D-30163 Hannover - Germany

Phone: +49-511-6862 8630 - Fax: +49-511-6862 8633

European Authorized Representative
Medical Device Safety Service

Reg. No. DE/CA0910A70122A

ENGLISH

Intellect[®] **TENS**



DUAL CHANNEL STIMULATOR KIT
A MAJOR STEP IN PAIN MANAGEMENT

INSTRUCTION MANUAL

CONTENTS

Indicators and Controls	2
Prescribing Information	3
Introduction	4
Preparation for Use	5
Instructions for Use.....	7
Concluding Treatment	12
Care and Maintenance	13
Troubleshooting	15
Technical Specifications	16

INDICATORS AND CONTROLS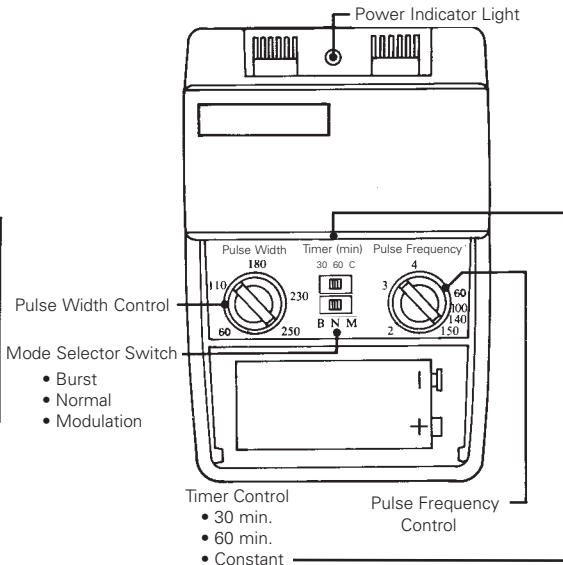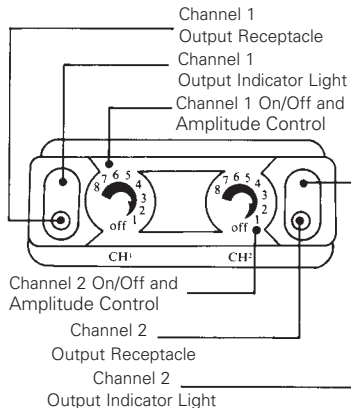

PRESCRIBING INFORMATION

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician. Keep out of reach of children.

Indications: Transcutaneous Electric Nerve Stimulation (TENS) devices are used for the symptomatic relief and management of chronic (long-term) intractable pain and as part of treatment in the management of acute pain problems after surgery or trauma.

Contraindications: TENS devices can affect the operation of cardiac pacemakers. In patients with known myocardial disease or arrhythmias, TENS should be used only with consultation and evaluation by a physician. Do not use TENS on the eyelids. Do not place electrodes directly over the carotid sinus nerves or arteries, and laryngeal or pharyngeal muscles. Do not apply TENS for undiagnosed pain syndromes until etiology is established. Any electrode placement that causes current to flow transcranially (through the head) is not allowed.

Warnings: This device should be used only under the continued supervision of a physician. TENS is ineffective for pain of central origin (in the central nervous system). TENS is of no curative value but is a symptomatic treatment which suppresses pain sensation which would otherwise serve as a protective mechanism on the outcome of the clinical process. Safety of TENS devices for use during pregnancy or delivery has not been established. For external use only. Electronic equipment such as EKG monitors and EKG alarms may not operate properly when TENS is in use.

Precautions: Avoid adjusting controls while operating machinery or vehicles. Turn the stimulator off before applying or removing electrodes. Long term stimulation at the same electrode site may cause skin irritation. If TENS therapy becomes ineffective or unpleasant, stimulation should be discontinued until its use is re-evaluated by the physician or therapist. Use only for the specific pain problem prescribed by physician. Effectiveness is dependent upon patient selection.

Adverse Reactions: Possible allergic reaction to tape or gel. Possible skin irritation or electrode burn under electrode. If skin irritation persists, discontinue use and consult a physician.

INTRODUCTION:

What is Pain: Pain warns our body of injury. This warning system is intended to prevent additional damage. The sensation of pain is important, because without it, vital parts of our bodies might be injured without our knowledge. However, long lasting and persistent pain, often called chronic pain, once diagnosed, serves no apparent purpose. TENS is developed to help relieve certain types of chronic and acute pain.



How Does TENS Work? TENS or Transcutaneous Electric Nerve Stimulation refers to the transmission of small electrical pulses through the skin to the underlying peripheral nerves. The theory of TENS suggests two different modes of operation:

Conventional (high frequency) TENS is based on the theory that continuous mild electrical activity may block the pain signal traveling to the brain. If the pain signal does not get through to the brain, the pain is not "felt."


The second way TENS is thought to work is by stimulating the body's own natural pain-control mechanism. "Low frequency" or short bursts of electrical activity may cause the body to release its own pain relieving substances, called endorphins. Ask your physician or therapist for more details. No matter what pain theory is used, TENS has been proven useful in pain management for many pain patients.

PREPARATION FOR USE

1. **Check Battery.** Ensure that you are using a fresh battery.
2. **Prepare Skin.** Before applying electrodes, be sure to confirm correct electrode placement as recommended by your physician or therapist. The skin should be washed, rinsed and thoroughly dried.
3. **Attach Electrodes.** Be sure that that the electrode is held firmly against the skin.

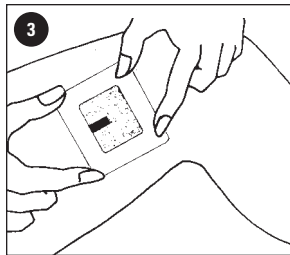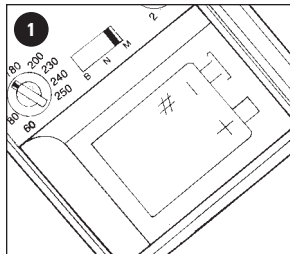
4. *Electrode Lead Wires*. Output Plug: (Plugs into output receptacle). Pin Connectors: (Plug into electrodes).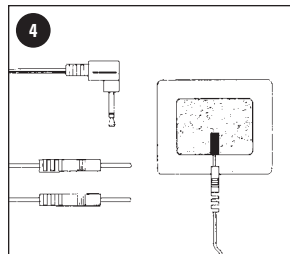


INSTRUCTIONS FOR USE


5. *Insert Pin into Electrode Socket.* Insert pin connector into electrode as shown. (When inserting or removing pin connector, hold connector, not the cord, to protect cord). For some electrode locations, it may be preferable to insert the pin connector into the electrode prior to taping the electrode to the skin.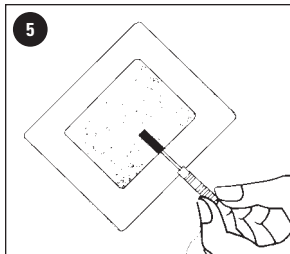


6. *Set Timer Selector.* 30 min., 60 Min. and Constant time.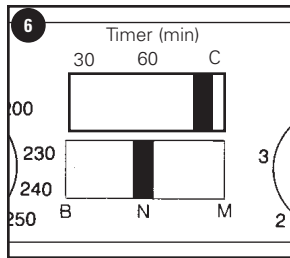


7. *Adjusting the Controls.* Ensure that amplitude controls for both Channels 1 and 2 are turned to the "OFF" position.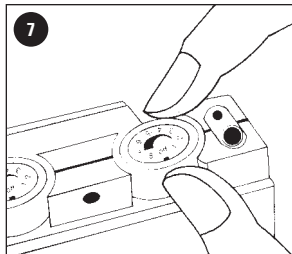


8. *Connect Leads to TENS Unit.* Insert 90° end of lead wire set into the channel output receptacle to be used (1 and /or 2), pushing plug all the way in.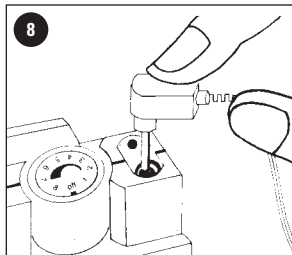


9. *High or Low Frequency Tens.* Set the mode selector switch to the "N" position.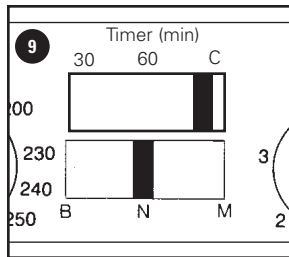


10. *Adjust Pulse Frequency.* Turn the Pulse Frequency control to the setting recommended by your medical professional.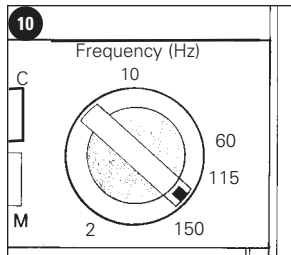


11. **Adjust Pulse Width.** Turn the Pulse Width control to the setting recommended by your medical professional.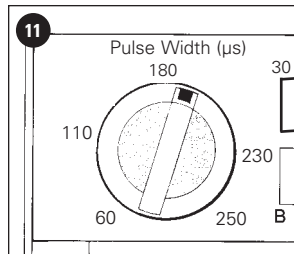


12. **Adjust Output.** Turn Amplitude control knob for Channel 1 or 2 clockwise. The indicator will light up while the unit is in operation. Slowly turn the channel control in a clockwise direction until you reach the setting recommended by your medical professional. Repeat for the other channel, if both channels are to be used.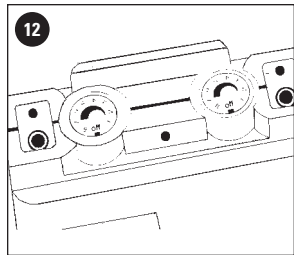

13. **Using Burst Mode.** When operating in Burst mode, your TENS produces a series of seven quick pulses followed by a rest. The cycle repeats itself twice every second. Before switching to Burst, turn amplitude controls for channels 1 and 2 to "O" (off). Choose "B" (burst mode) with the mode selector switch. Slowly adjust the amplitude and pulse width controls to the setting recommended by your medical professional. Note: In burst mode, the pulse frequency is fixed; adjusting the pulse frequency control has no effect.

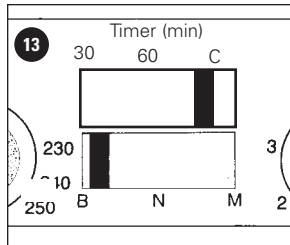
14. **Using Modulation Mode.** When operating in Modulation Mode, your TENS unit automatically varies the amplitude of the pulse every four seconds. To use Modulation, first adjust the amplitude controls for channels 1 and 2 to "O" (off). Set the mode selector switch to "M" (modulation) position. Slowly adjust channel 1 and/or channel 2 amplitude, pulse frequency and pulse width controls to the settings recommended by your medical professional.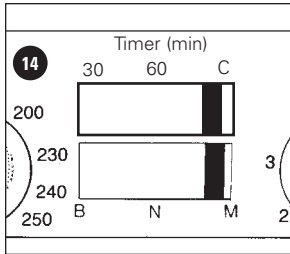


CONCLUDING TREATMENT

15. *Turning Unit Off.* Turn both channel controls to Off. Then unplug the electrode lead wires, grasping them by the plug, not the cord. If treatment will be resumed shortly, the electrodes may be left on the skin. When the electrodes are removed, clean the skin. If there is skin irritation, consult your physician or therapist.

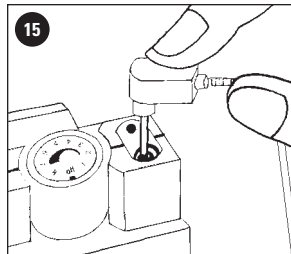
16. *Portability.* Your TENS is portable and may be clipped to a belt, shirt pocket, bra, or other clothing.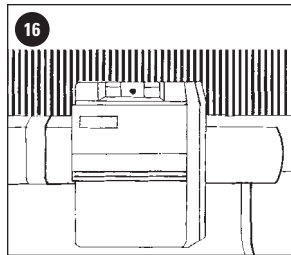


CARE AND MAINTENANCE


17. **Front Cover.** A removable panel covers the controls for pulse frequency, pulse width, mode switch and battery compartment. Your medical professional may wish to set these controls for you and request that you leave the cover in place.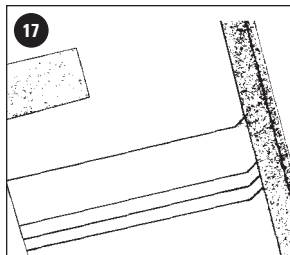

18. **Battery.** Dimming of the Indicator lights signifies that the battery should be replaced with a new one as soon as possible. However, the stimulator will continue to operate for several more hours. To replace battery, remove front cover (see above) and extract battery. Replace with 9-volt alkaline or similar rechargeable battery, taking care that the battery is inserted correctly. (See diagram inside battery compartment). This TENS unit is designed to be used with batteries only and cannot be operated from a line-powered battery charger.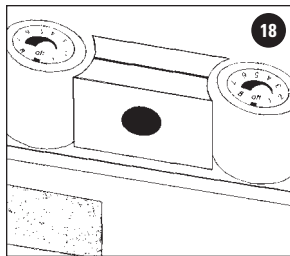

19. *Care of Electrode Cords.* Clean the electrode cords by wiping with a damp cloth. Coating them lightly with talcum powder will reduce tangling and prolong life.

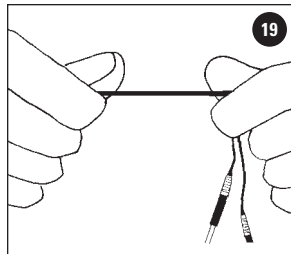TROUBLESHOOTING

If your TENS unit does not seem to be operating correctly, refer to the chart below to determine what may be wrong. Should none of these measures correct the problem, the unit will need servicing.

The power indicator lights up but unit does not function properly.

Please turn both amplitude control switches off, then turn on again.

Check all control settings. Are they set to values prescribed by your medical professional?

Are electrodes in proper position?

Check lead wires. Be sure all connectors are firmly sealed.

Be sure you are using the proper amount of gel. Too much or too little can cause problems.

Replace cord set with another to check for broken wires.

"On" and "Battery Light" are dim.

Replace battery.

None of the indicators light up.

Replace battery.

To obtain service, contact your supplier.

TECHNICAL SPECIFICATIONS

<i>Channels:</i>	Dual, isolated between channels
<i>Wave Form:</i>	Modified square wave with zero net direct current (DC) component.
<i>Pulse Amplitude:</i>	0 to 80 mA each channel, adjustable (500 ohm load)
<i>Pulse Frequency:</i>	2 to 150 HZ., adjustable
<i>Pulse Width:</i>	60 to 250 microseconds, adjustable
<i>Modulation Mode:</i>	Pulse width is automatically varied in a cyclic pattern over an interval of nominally 4.0 seconds.
<i>Burst Mode:</i>	7 pulses per burst 2 bursts per second 100 Hz internal frequency of burst
<i>Maximum Charge:</i>	20 micro coulombs per pulse
<i>Timer:</i>	30 Min, 60 Min or Constant time
<i>Power Source:</i>	9-volt alkaline battery or similar rechargeable cell
<i>Battery Life:</i>	70 hours average usage (alkaline)
<i>Size:</i>	24 x 64 x 95 mm
<i>Weight:</i>	130 grams (including battery)

All values have 10% ± tolerance.

ATTENTION: Consult accompanying documents

WARNING: Keep electrodes separated during treatment.
Electrodes in contact with each other could result in improper stimulation or skin burns.

Type BF equipment.

Internally Powered Equipment

Care and Maintenance: Periodically (or as necessary) clean with slightly damp cloth and mild soap. *(No Protection Against Harmful Ingress of Water).*

Meets IEC/ EN 60601-1-2 Electromagnetic Compatibility/ interference safety standard. *(Care must be taken when operating this equipment around other equipment. Potential electromagnetic or other interference could occur to this or to the other equipment. Try to minimize this interference by not using other equipment in conjunction with it).*

Dispose of used batteries in an environmentally safe manner. *(Remove batteries during extended periods of non-use).*

Device is Not User Serviceable *(Return to manufacturer for repair).*

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© 2003 Encore Medical
1-423-870-2046 U.S.A.