



# TeWa: The Future of Acupuncture

## Introducing a Breakthrough in Guide Tube Technology - the Safe-T Sleeve™


### How to use the Safe-T Sleeve™

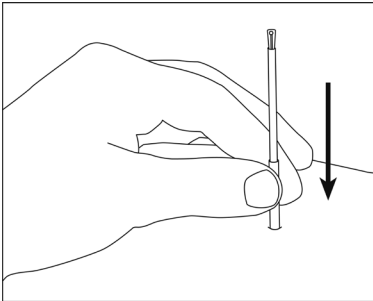


STEP ONE: Remove plastic tab and slide Safe-T Sleeve™ down so it's flush with the bottom of the insertion tube.

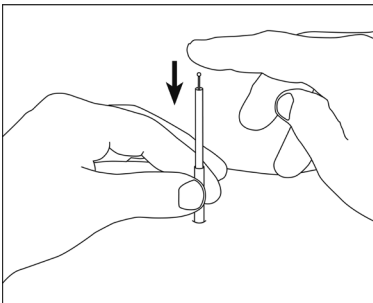

STEP TWO: Tap needle into acupoint as usual.

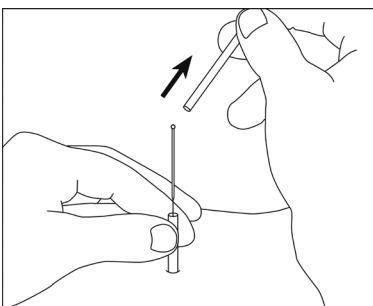

STEP THREE: While holding the Safe-T Sleeve™ in place, remove insertion tube with other hand.

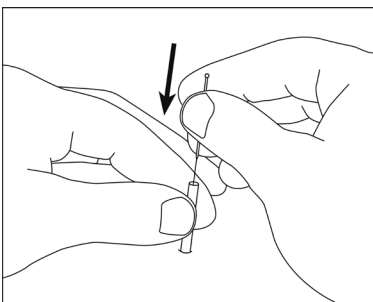

STEP FOUR: Guide the needle to the target depth while supporting the needle shaft with the Safe-T Sleeve™. The Safe-T Sleeve™ can then be removed or left in place.



Available exclusively on TeWa needles, the disposable Safe-T Sleeve™ ensures clean needling every time.

The Safe-T Sleeve™ was created to solve the problems that arise when inserting longer needles into dense or tight tissue - namely the potential bowing of the needle and difficulty stabilizing the needle shaft.

Now you can utilize the Safe-T Sleeve™ to help complete insertion to the target depth - all without compromising the sterility of the needle shaft.

The flexible, two-part guide tube system is perfect for sports acupuncture, Motor Point Needling, and other advanced needling techniques.

The Safe-T Sleeve™ is available exclusively on TeWa ("Tay-Vah") coated and non-coated needles 1.5" or longer.

Check out [www.safe-t-sleeve.com](http://www.safe-t-sleeve.com) for instructional videos demonstrating the Safe-T Sleeve™ and its use in general needling, sports acupuncture, and other specialty techniques!

◁ A TeWa needle with the insertion tube removed and the Safe-T Sleeve™ in place.



**Safe-T Sleeve™**  
PATENT PENDING

DISTRIBUTED BY



190, Boul. St-Elzéar Ouest  
Laval (Québec) H7L 3N3  
T 450 669.8985 | 1 800 361.0877  
F 450 669.9532 | 1 888 935.7001

[www.medicaltronik.ca](http://www.medicaltronik.ca)

# TeWa: A new clinic standard

Looking for an affordable needle that's smooth, super sharp, and versatile?  
Look no further! New TeWa needles offer you:

- Copper wire wound-loop handles
- Needle tips sharpened by machine, ensuring perfect needle tip geometry
- Available with coated needle shafts for super smooth insertion (non-coated needles also available)
- Enhanced features: The Safe-T Sleeve™ comes standard on all needles 40mm and longer - it's there when you need it!



TeWa Coated needles	
Code	Description
TWCJ1615	TeWa Coated .16 x 15mm (40 - 0.6")
TWCJ1630	TeWa Coated .16 x 30mm (40 - 1.2")
TWCJ1815	TeWa Coated .18 x 15mm (38 - 0.6")
TWCJ1830	TeWa Coated .18 x 30mm (38 x 1.2")
TWCJ1840	TeWa Coated .18 x 40mm (38 - 1.5") w Safe-T Sleeve™
TWCJ2015	TeWa Coated .20 x 15mm (36 - 0.6")
TWCJ2030	TeWa Coated .20 x 30mm (36 - 1.2")
TWCJ2040	TeWa Coated .20 x 40mm (36 - 1.5") w Safe-T Sleeve™
TWCJ2050	TeWa Coated .20 x 50mm (36 - 2.0") w Safe-T Sleeve™
TWCJ2213	TeWa Coated .22 x 13mm (34 - 0.5")
TWCJ2225	TeWa Coated .22 x 25mm (34 - 1.0")
TWCJ2240	TeWa Coated .22 x 40mm (34 - 1.5") w Safe-T Sleeve™
TWCJ2250	TeWa Coated .22 x 50mm (34 - 2.0") w Safe-T Sleeve™
TWCJ2275	TeWa Coated .22 x 75mm (34 - 3.0") w Safe-T Sleeve™
TWCJ2515	TeWa Coated .25 x 15mm (32 - 0.6")
TWCJ2525	TeWa Coated .25 x 25mm (32 - 1.0")
TWCJ2530	TeWa Coated .25 x 30mm (32 - 1.2")
TWCJ2540	TeWa Coated .25 x 40mm (32 - 1.5") w Safe-T Sleeve™
TWCJ2550	TeWa Coated .25 x 50mm (32 - 2.0") w Safe-T Sleeve™
TWCJ2575	TeWa Coated .25 x 75mm (32 - 3.0") w Safe-T Sleeve™
TWCJ3015	TeWa Coated .30 x 15mm (30 - 0.6")
TWCJ3025	TeWa Coated .30 x 25mm (30 - 1.0")
TWCJ3040	TeWa Coated .30 x 40mm (30 - 1.5") w Safe-T Sleeve™
TWCJ3050	TeWa Coated .30 x 50mm (30 - 2.0") w Safe-T Sleeve™
TWCJ3075	TeWa Coated .30 x 75mm (30 - 3.0") w Safe-T Sleeve™
TWCJ30100	TeWa Coated .30 x 100mm (30 - 4.0") w Safe-T Sleeve™
TWCJ3575	TeWa Coated .35 x 75mm (28 - 3.0") w Safe-T Sleeve™
TWCJ35100	TeWa Coated .35 x 100mm (28 - 4.0") w Safe-T Sleeve™

TeWa Non-Coated needles	
Code	Description
TWNJ1630	TeWa Non-Coated .16 x 30mm (40 - 1.2")
TWNJ2013	TeWa Non-Coated .20 x 13mm (36 - 0.5")
TWNJ2025	TeWa Non-Coated .20 x 25mm (36 - 1.0")
TWNJ2040	TeWa Non-Coated .20 x 40mm (36 - 1.5") w Safe-T Sleeve™
TWNJ2050	TeWa Non-Coated .20 x 50mm (36 - 2.0") w Safe-T Sleeve™
TWNJ2213	TeWa Non-Coated .22 x 13mm (34 - 0.5")
TWNJ2225	TeWa Non-Coated .22 x 25mm (34 - 1.0")
TWNJ2240	TeWa Non-Coated .22 x 40mm (34 - 1.5") w Safe-T Sleeve™
TWNJ2250	TeWa Non-Coated .22 x 50mm (34 - 2.0") w Safe-T Sleeve™
TWNJ2275	TeWa Non-Coated .22 x 75mm (34 - 3.0") w Safe-T Sleeve™
TWNJ2513	TeWa Non-Coated .25 x 13mm (32 - 0.5")
TWNJ2525	TeWa Non-Coated .25 x 25mm (32 - 1.0")
TWNJ2540	TeWa Non-Coated .25 x 40mm (32 - 1.5") w Safe-T Sleeve™
TWNJ2550	TeWa Non-Coated .25 x 50mm (32 - 2.0") w Safe-T Sleeve™
TWNJ2575	TeWa Non-Coated .25 x 75mm (32 - 3.0") w Safe-T Sleeve™
TWNJ3013	TeWa Non-Coated .30 x 13mm (30 - 0.5")
TWNJ3025	TeWa Non-Coated .30 x 25mm (30 - 1.0")
TWNJ3040	TeWa Non-Coated .30 x 40mm (30 - 1.5") w Safe-T Sleeve™
TWNJ3050	TeWa Non-Coated .30 x 50mm (30 - 2.0") w Safe-T Sleeve™
TWNJ3075	TeWa Non-Coated .30 x 75mm (30 - 3.0") w Safe-T Sleeve™
TWNJ30100	TeWa Non-Coated .30 x 100mm (30 - 4.0") w Safe-T Sleeve™
TWNJ3575	TeWa Non-Coated .35 x 75mm (28 - 3.0") w Safe-T Sleeve™
TWNJ35100	TeWa Non-Coated .35 x 100mm (28 - 4.0") w Safe-T Sleeve™

DISTRIBUTED BY



190, Boul. St-Elzéar Ouest  
Laval (Québec) H7L 3N3  
T 450 669.8985 | 1 800 361.0877  
F 450 669.9532 | 1 888 935.7001  
[www.medicaltronik.ca](http://www.medicaltronik.ca)