Upholstery

PHSChiropractic by Pivotal Health Solutions

Stock Vinyl Uphostery Colors upholstery care and cleaning guide

For light soiling, use a solution of 10% household liquid dish soap with warm water, applied with soft damp cloth. Rinse with clean water and dry. For heavy soiling, dampen a soft white cloth in a one to one (1:1) solution of Fantastik® and water OR Formula 409® and water. Rub gently and rinse with a water dampened cloth. For more difficult stains, dampen a soft white cloth with a solution of household bleach (10% bleach / 90% water). Rub gently and rinse with a water dampened cloth to remove bleach concentration. DO NOT USE ALCOHOL BASED CLEANING AGENTS! IMPROPER CLEANING WILL VOID THE WARRANTY. This information is not a guarantee and does not relieve the user from the responsibility of the proper and safe use of the product and all cleaning agents. The use of certain agents can be harmful to the surface appearance and lifespan of vinyl. Pivotal Health Solutions assumes no responsibility resulting from the use of such agents to the vinyl.

