

ALLFIT™

Allfit™ always fits



Allfit™ always fits your exact needs. And those of your patient.



1. Simply pour the pellets into hot water (65°C - 149°F). When they have transformed into a transparent mass, the product is ready to use.



2. Now the mass can be hand-moulded into any imaginable shape.



3. You now can apply Allfit™ to change the shape of an appliance, utensil or any other object. This example shows an improvement of the patient's grip function.



4. You can imagine any application: enlarge, enhance, rend heavier, reshape and change. Your patient's needs are your guide...

ALLFIT™

Box with 6 bags of 500 g

703011